


ASSEMBLY OF ACCESSORY CONNECTION FOR COLONOSCOPE/GASTROSCOPE WITH NEW SILICON FITTINGS & BRUSHES (example Olympus ETAK01 Accessory Kit)

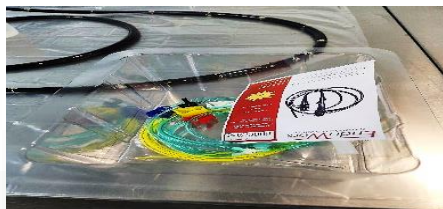

1. Open Accessory Kit

Ensure package is unbroken and EO indicator is active.

- A. Empty contents into in clear base from collection set.
- B. **Move pouch with brush(s) aside for brushing sequence, do not open till required**



2. Water Supply

Insert water line into water bottle using bottle clip. **Ensure not to touch bottle line approx. 200mm from end (keep line area sterile). Ensure bottle is close to scope to minimise tension on line.**



3. Suction Line Yellow Clamp Line

Attach assembly to suction port on light guide connector ensuring fitting is seated over connection.



4. Biopsy Line White Clamp Line

Attach biopsy fitting to biopsy port located on control handle ensuring fitting is seated over connection



5. Air Inlet Connection Red Clamp Line

Attach tube to air inlet line below light guide post
Ensure tube is inserted correctly.



6. Auxiliary Connection Blue Clamp Line

Place silicon fitting over the auxiliary port **with a push and twist motion** ensuring the fitting is seated past the ridges of the auxiliary port

(Non-Aux. scopes skip this step, clamp closed line)



7. Attach Water Bypass

Attach water bypass to light guide connector, ensuring fitting is seated correctly



8. Cylinder Caps

Place caps over suction and air/water cylinders ensuring caps are seated correctly ensure not to touch inside of caps.