
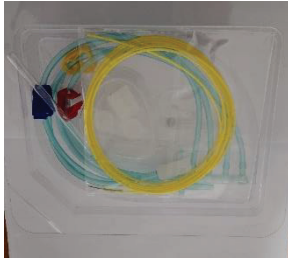


ASSEMBLY OF ACCESSORY KIT FOR PENTAX COL & GAS ENDOSCOPES

Step 2. Connecting the lines to each port on the scope using the clamp colours to determine the port type.
Maneuver the sterile water bottle, with lines closer to the scope control head to access ports easier.

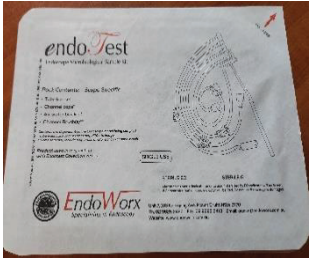

1. Open Accessory Set
2. Ensure package is unbroken and EO indicator is active.
3. Peel back lid
Move brush(s) aside till brushing sequence. (Brush colour may differ dependant on accessory kit)

2. Water Supply

Ensure your sterile bottle of water is in position for easy access. Pick up manifold at the bottle clip section (**ensuring you do not touch the line that enters the bottle**) insert the hard tube section into the bottle and gently attach the bottle clip to the rim to secure the line in the sterile water.

3. Suction Line Yellow Clamp Line

Place silicon fitting onto suction proximal connector ensuring the fitting is seated over the ridges of the suction fitting.


4. Biopsy Line White Clamp Line

Attach Luer fitting to the biopsy with a twist to lock into place



5. Air Water Connection Red Clamp Line

Place tube line with red clamp to air water channel firmly pushing plug until seated

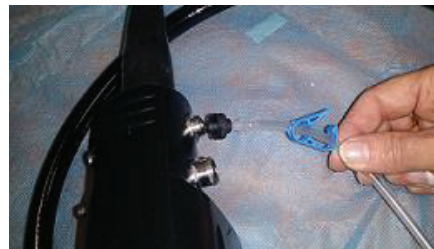

6. Forward Flushing Connection Blue Clamp Line

Place tube line with blue clamp firmly pushing plug until seated



8. Cylinder Caps

Place caps over suction and air/water cylinders Ensure caps are seated correctly ensure not to touch inside of caps or cylinder surface



Follow step 3. colour charts for collection steps relating to scope model