



ASSEMBLY OF ACCESSORY KIT FOR OLYMPUS COL & GAS ENDOSCOPES

Step 2. Connecting the lines to each port on the scope using the clamp colours to determine the port type.
Maneuver the sterile water bottle, with lines closer to the scope control head to access ports easier.

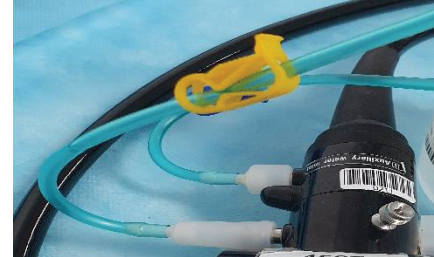


1. Open Accessory Set
2. Ensure package is unbroken and EO indicator is active.
3. Peel back lid
Move brush(s) aside till brushing sequence. (Brush colour may differ dependant on accessory kit)

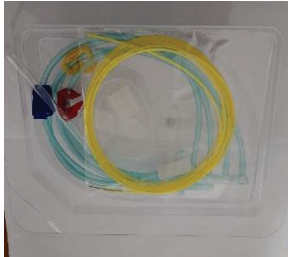

5. Air Inlet Connection Red Clamp Line
Attach tube to air inlet line below light guidepost. Ensure tube is inserted correctly.

2. Water Supply

Ensure your sterile bottle of water is in position for easy access. Pick up manifold at the bottle clip section (**ensuring you do not touch the line that enters the bottle**), insert the hard tube section into the bottle and gently attach the bottle clip to the rim to secure the line in the sterile water.

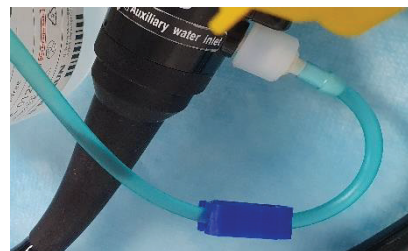

6. Auxiliary Connection Blue Clamp Line

Place silicon fitting over the auxiliary port **with a push and twist motion** ensuring the fitting is seated past the ridges of the auxiliary port.

Clamp off if no Aux channel on scope.

3. Suction Line Yellow Clamp Line

Place silicon fitting onto suction proximal connector ensuring the fitting is seated over the ridges of the suction fitting.



7. Attach Water Bypass

Attach the silicon blocker to the water bottle connector. Start with small hole then large to ensure it is seated flush against scope.

4. Biopsy Line White Clamp Line

Place silicon fitting over the biopsy port ensuring the fitting is seated firmly over the biopsy port fitting



8. Cylinder Caps

Place caps over suction and air/water cylinders. Ensure caps are seated correctly ensure not to touch inside of caps or cylinder surface.

Follow step 3. colour charts for collection steps relating to