


## ASSEMBLY OF ACCESSORY KIT FOR BRONCHOSCOPES, BALLOON & CYSTOSCOPES

**Step 2. Connecting the lines to each port on the scope using the clamp colours to determine the port type.**  
**Maneuver the sterile water bottle, with lines closer to the scope control head to access ports easier.**



Refer to following images relating to the suction fitting type

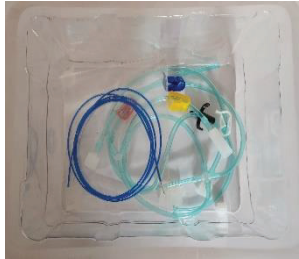

### 1. Open Accessory Set

Ensure package is unbroken and EO indicator is active.  
 Move brush(s) aside till brushing sequence. (Brush colour may differ dependant on accessory kit)

### 2. Water Supply

Ensure your sterile bottle of water is in position for easy access. Pick up manifold at the bottle clip section (**ensuring you do not touch the line that enters the bottle**), insert the hard tube section into the bottle and gently attach the bottle clip to the rim to secure the line in the sterile water.



### 3. Suction Line (Bronchoscope/Cystoscopes) Yellow Clamp Line

Place silicon fitting into or over suction port ensuring it forms a tight fit.



### 4. Storz Scopes Suction/Biopsy

Connect both yellow/white luer fitting lines as per image

### 5A. Balloon Channel (UC180/190)

Connect the blue clamp tube line by attaching the luer lock fitting to the balloon channel.

### 5B. Balloon Channel Pentax

Connect the blue clamp tube line by attaching the silicon balloon fitting to the balloon channel.

### 6A. Biopsy Line White Clamp Line

Place silicon fitting over the biopsy port ensuring the fitting is seated firmly over the biopsy port fitting

### 6B. Biopsy Line White Clamp Line

Connect the white clamp tube line by connecting the luer fitting to the biopsy port connector