
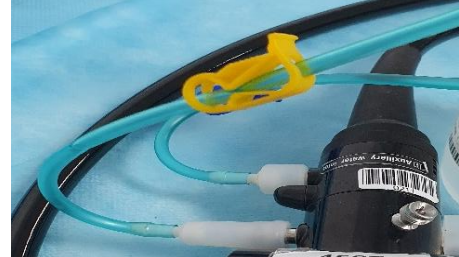
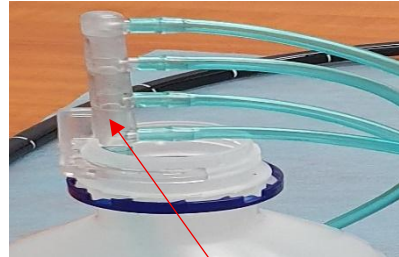


Chart 2. Connecting the lines to each port on the scope using the clamp colours to determine the port type.
Maneuver the sterile water bottle, with lines closer to the scope control head to access ports easier.

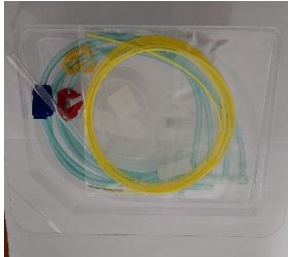

1. Open Accessory Set

Ensure package is unbroken and EO indicator is active.

Peel back lid
 Move brush(s) aside till brushing sequence.

2. Water Supply

Ensure your sterile bottle of water is in position for easy access. Pick up manifold at the bottle clip section (**ensuring you do not touch the line that enters the bottle**) insert the hard tube section into the bottle and gently attach the bottle clip to the rim to secure the line in the sterile water.


33. Suction Yellow Clamp Line

Place silicon fitting onto suction proximal connector ensuring the fitting is seated over the ridges of the suction fitting.

4. Biopsy White Clamp Line

Attache the luer lock fitting to the biopsy port.

- Dual channel scopes connect both biopsy lines.

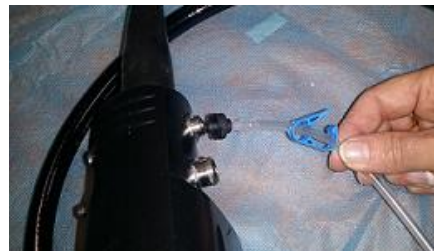

5. Air Inlet Connection Red Clamp Line

Place tube line with red clamp to air water channel firmly pushing plug until seated

6. Forward Flushing Connection Blue Clamp Line

Place silicon fitting over the auxiliary port ensuring the fitting is seated correctly



8. Cylinder Caps (now white)

Place caps over suction and air/water cylinders. Ensure caps are seated correctly ensure not to touch inside of caps or cylinder surface

Follow chart 3. colour charts for collection steps relating to scope type