


Chart 2. Connecting the lines to each port on the scope using the clamp colours to determine the port type.
Maneuver the sterile water bottle, with lines closer to the scope control head to access ports easier.

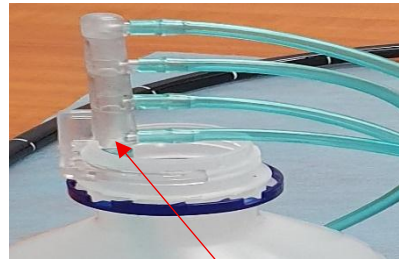

1. Open Accessory Set

Ensure package is unbroken and EO indicator is active.

Peel back lid
 Move brush(s) aside till brushing sequence.

2. Water Supply

Ensure your sterile bottle of water is in position for easy access. Pick up manifold at the bottle clip section (**ensuring you do not touch the line that enters the bottle**), insert the hard tube section into the bottle and gently attach the bottle clip to the rim to secure the line in the sterile water

3. Suction Line Yellow Clamp Line

Place silicon fitting into suction proximal connector ensuring the fitting is seated in the suction port.

For luer lock fittings attached luer fitting to port.

4. Biopsy Line White Clamp Line

Silicon fitting over the biopsy port ensuring the fitting is seated firmly over the biopsy port fitting.

For luer lock fittings attached luer fitting to port.

4. Fujinon/Storz Suction Yellow Clamp

Place silicon fitting into suction proximal connector ensuring the fitting is seated in the suction port.



4. Storz Suction/Biopsy Luer Fittings

Connect both yellow/white luer fitting lines as per image

Follow chart 3. colour charts for collection steps relating to the scope model