
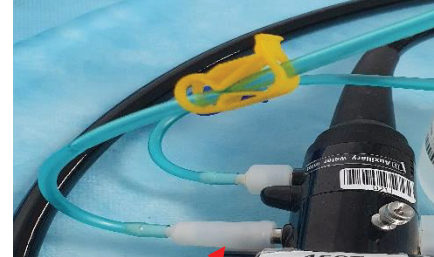


ASSEMBLY OF ACCESSORY KIT FOR FUJINON COL & GAS ENDOSCOPES

Step 2. Connecting the lines to each port on the scope using the clamp colours to determine the port type.
Maneuver the sterile water bottle, with lines closer to the scope control head to access ports easier.

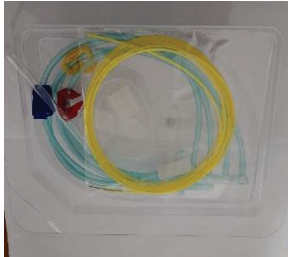

1. Open Accessory Set

Ensure package is unbroken and EO indicator is active.

2. Peel back lid

Move brush(s) aside till brushing sequence. (Brush colour may differ dependant on accessory kit)

2. Water Supply


Ensure your sterile bottle of water is in position for easy access. Pick up manifold at the bottle clip section (**ensuring you do not touch the line that enters the bottle**), insert the hard tube section into the bottle and gently attach the bottle clip to the rim to secure the line in the sterile water.

3. Suction Line Yellow Clamp Line

Place silicon fitting onto suction proximal connector ensuring the fitting is seated over the ridges of the suction fitting.

4. Biopsy Line White Clamp Line

Place silicon fitting over the biopsy port ensuring the fitting is seated firmly over the biopsy port fitting

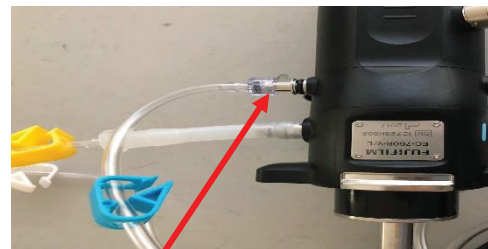

5. Air Inlet Connection Red Clamp Line

Attach tube to air inlet line below light guidepost. Ensure tube is inserted correctly.

6. Auxiliary Connection Blue Clamp Line

Attach luer fitting assembly to auxiliary port

7. Attach Water Bypass

Attach supplied bung to water bypass, ensuring fitting is seated correctly

8. Cylinder Caps

Place caps with tabs over suction cylinder and plug style cap over air/water cylinder ensuring caps are seated correctly ensure not to touch inside of caps or cylinder surface

Follow step 3. colour charts for collection steps relating to scope model